Utensil Strap

VM913

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Utensil Strap from Aidapt. The Utensil Strap is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

Designed for use with these products from Aidapt:

VM914 Bendable Cutlery Set

VM911 Bendable Fork

VM910 Bendable Spoon

VM912 Bendable Rocker Knife

BEFORE USE

Carefully remove all packaging. Avoid using any knives of other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

The Utensil Strap has been designed to fit the Dining Aid range from Aidapt; including the Bendable Spoon, Bendable Fork and Rocker Knife. The Utensil Strap provides a secure way to hold the utensil, particularly for those with limited grip or weakness in the hand and arm.

The Utensil Strap features a length of strong webbing with a loop on one end and a Velcro fastener on the other.

INSTRUCTIONS FOR USE

- Insert the handle of the Utensil through the loop of the Utensil Strap, pushing the loop down to the base of the handle closest to the metal utensil.
- Open the Velcro fastener on the Utensil Strap and slip it down through the loop at the end of the utensil, bringing it back onto itself to fasten.
- The utensil can now be used as normal, with the Utensil Strap over the back of the hand.
- Assistance may be needed to further tighten the Utensil Strap to ensure a snug fit for effective use.

CLEANING

Clean your Utensil Strap using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

Usage and Maintenance Instructions

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Altai Germany (Altai Europe Ltd.)