## **Egg Cups (Pair)**

**VM908**

# **Usage and Maintenance Instructions**

## **INTRODUCTION**

Thank you for deciding to purchase the Egg Cups from Aidapt. The Egg Cups are manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

## **BEFORE USE**

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## **INTENDED USE**

These useful Egg Cups have a secure, non-slip suction base to grip a hard, flat surface such as a table, trolley or lap tray and are ideal for those with the use of only one hand, assisting independent dining.

## **INSTRUCTIONS FOR USE**

- Ensure that the surface on which you will be positioning the Egg Cup is clean and dry.
- Position the Egg Cup as desired then press down to secure it to the surface with the built-in non-slip suction base.
- To remove the plate break the suction by raising one edge slightly and lifting the suction base, then twist the base to fully release the suction and lift it free from the surface.

**NB. If the Egg Cups are not empty, care must be taken when removing them from the surface to avoid spillage.**

## **CLEANING**

Clean your Egg Cup using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you rinse then wipe dry the equipment after cleaning.

The Egg Cup can also be cleaned on the top tray of a domestic dishwasher on a standard/eco or cool program.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

Usage and Maintenance Instructions

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**Altai Germany (Altai Europe Ltd.)**