Shower Chair with Back

VB540S

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Shower Chair with Back from Aidapt. When used correctly it is designed to give many years of reliable trouble-free service.

NB. This equipment must be installed by a competent person.

120 kg (18 ³/₄ stone) weight limit.

Do not exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier immediately for support.

PARTS LIST

NO.	DESCRIPTION	SPECIFICATION	QTY
11	Spring Pin	Ø 8	4
10	Seat Board		1
9	Backrest Board		1
8	Screw Bolt	M6x30	4
7	Backrest Tube		1
6	Screw Bolt	M6x35	1
5	Arch Washer	Ø 6	9
4	Screw Bolt	M6x55	4
3	Extendable Tubes		4
2	Lower Tube		1
1	Upper Tube		1

ASSEMBLY PROCEDURE

1. Join the Upper and Lower Tubes as shown in Fig. 1.
2. Lay the seat upside down and put the joined Upper and Lower Tubes into the slot of the seat ensuring that marks correspond as per Fig. 2, then tighten all screws.
3. Fix the backrest tubes onto the backrest board as shown in Fig. 2, with the long part of the backrest tube attached to the backrest board.
4. Fix the shorter end of the tubes on the assembled backrest onto the assembled upper/lower tubes and seat.
5. Turn the assembled Shower Stool the right way up on a flat surface. Equally adjust each of the legs to the same height/number setting as per Fig. 3. Remember to check the spring pins before each use.

NB. We do not recommend the use of a battery operated screw driver when assembling this product.

ASSEMBLY DIAGRAM

Fig. 1

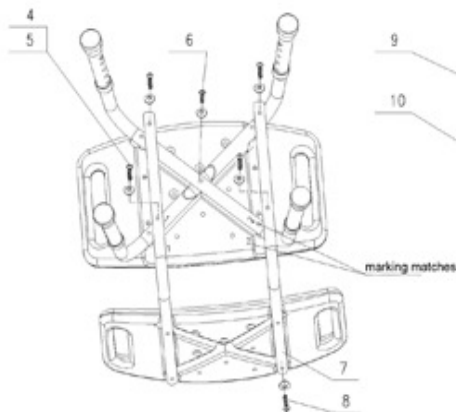
Fig. 2

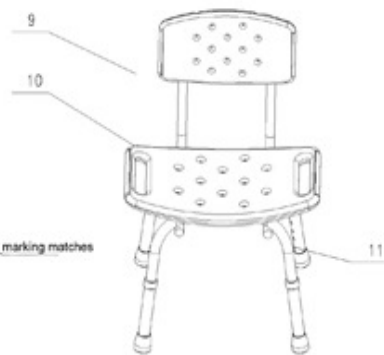
Fig. 3

HEIGHT ADJUSTMENT

1. Depress the securing pin and twist the extension leg to the side to allow you to slide the extension leg up or down to the desired height.
2. Rotate the leg back to allow the securing pin to pop out of the hole and fix the height at the correct setting.
3. Repeat steps 2.1 and 2.2 above for each leg, ensuring that all four legs are set to the same height setting.

CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower chairs and stools using a non-abrasive cleaner or mild detergent with a soft cloth, avoid using lime scale remover. Abrasive cleaners e.g. AJAX and or abrasive cleaning pads will seriously damage your product beyond repair.

Do not exceed 50°C when cleaning the seat and/or back as this will permanently damage the plastic.

N.B. Regular daily cleaning will reduce the buildup of limescale on your product.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. This includes:

- Bent or damaged tubes or cracks at the tube joints.
- Elongated adjustment holes.
- Loose or worn fasteners
- Smooth, worn or split ferrules (rubber tips).

Replace all worn, missing or damaged parts where necessary. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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**Altai Germany (Altai Europe Ltd.)